

## User guide


### BEFORE USE

- The battery must be loaded for at least 12 hours the first time
- Adjust the doses (PD and ILT) by testing with water \*


### AFTER USE

- Clean the equipment very well with water
- Fill the liquid depots with water while the equipment is not used !
- After several times of use it's recommended to clean very well with high pressure air. Make sure that the small holes in the covers of the liquid depots are not closed by dirty.

#### \* AJUSTING DOSES

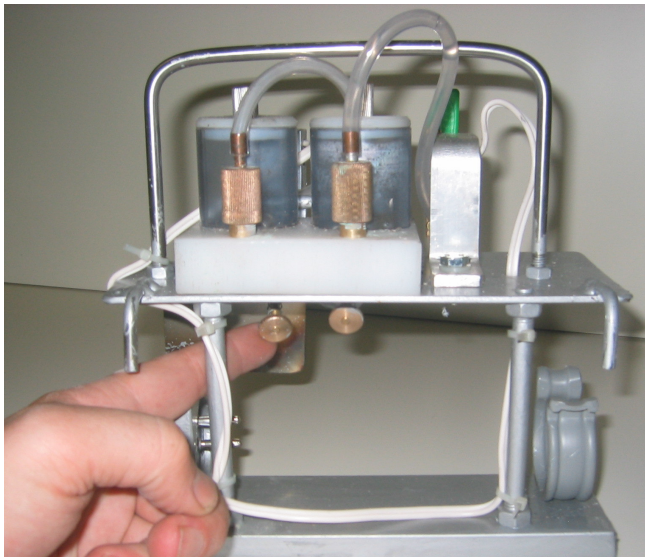


adjusting ILT-vaccination

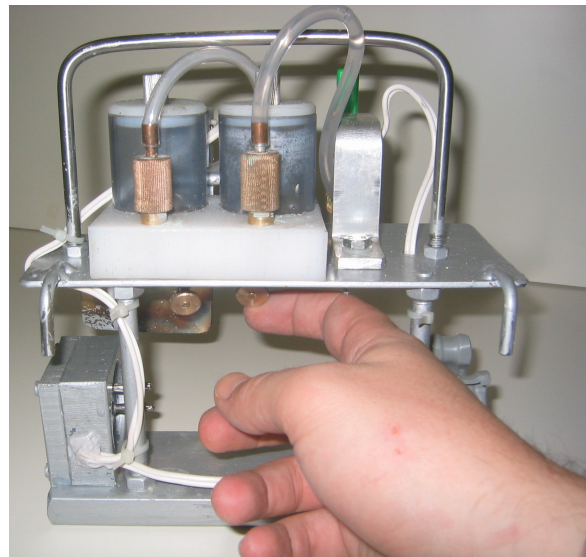

adjusting PD-enting



Cleaning

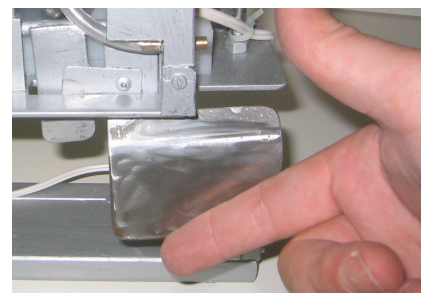

press to vaccinate

